



This guide is for iOS 10+ iPhone users.



AbiBird®

How to

Set or Change Notification Preferences


By setting your notification preferences you tell a sensor when to send you emails or push notifications.

To set or change notification preferences in the AbiBird app, you need to:

1. Access Notifications page
2. Turn notifications ON or OFF

Step 1: Access 'Notifications' page

The way you access the *Notifications* page will depend on whether you are setting up AbiBird for the first time or changing notification preferences that you have already set.


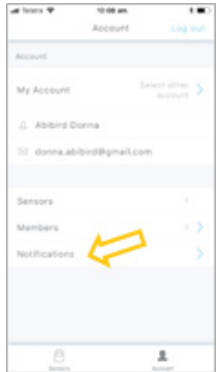
If...	Then...	
You are setting up AbiBird for the first time...	<p>The <i>Notifications</i> page opens after you have set up daily routines. (See guide: <i>How to Set Up or Modify Daily Routines</i>)</p> <p>By default, all notifications are OFF.</p> <p>[Go to Step 2]</p>	




This guide is for iOS 10+ iPhone users.



AbiBird®

If...	Then...	
You have already set up AbiBird and want to change your notification preferences...	<ul style="list-style-type: none">• Login to AbiBird with your username (email) and password. The sensor menu opens.• Select the Account icon (bottom right). The <i>Account</i> menu will open.	
	<ul style="list-style-type: none">• From the Account menu, select Notifications. The <i>Notifications</i> page will open. [Go to Step 2]	

Step 2: Turn notifications on or off

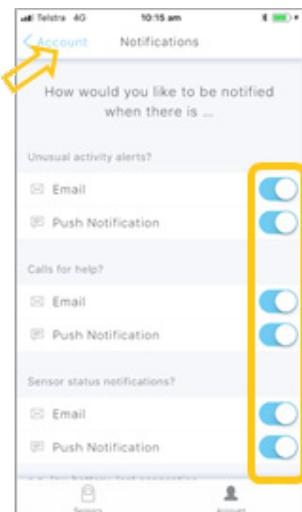

On the *Notifications* page, tap each 'switch' to select how you would like to be notified (Email and/or Push Notification) when any of your sensors detect:

- Unusual activity – when activity in the room doesn't match the routine you set
- A 'Call for Help' – when the sensor has been tapped or shaken
- A change to a sensor's status – when the battery is low or connection to the network has been lost

When you are done, select < **Account** (top left) to return to the Account menu. Your notification preferences will be saved.

The example shows all notifications turned ON




This guide is for iOS 10+ iPhone users.



AbiBird

That's it

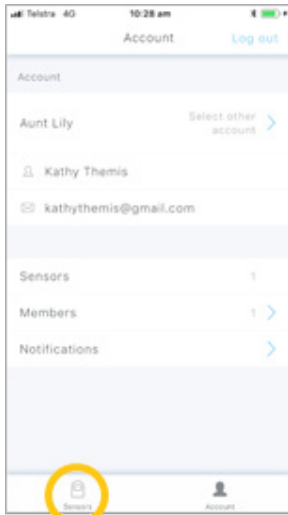

Notification preferences have been set.

NOTE: to return to the *Sensors* menu, select the **Sensors** icon (bottom left).

Working in the app, you have:

- Set how you would prefer to receive notifications from AbiBird for unusual activity, calls for help and changes to sensor/s status.

NOTE

Notification preferences apply to all sensors paired to your AbiBird account. However, you can pause notifications for a particular sensor (see guide: [How to Pause Sensor Notifications](#)).

Next Step

Your next step is to:

- Add another sensor if you have more than one sensor.

(See the guide: [How to Add Additional Sensors](#))

or

- Begin using AbiBird as initial setup is now complete.

You can:

- Check the sensor's status and rename it if required
(See guide: [How to Check Sensor Status and Rename Sensor](#))
- Modify the sensor's daily routine (See guide: [How to Set Up or Modify Daily Routines](#))
- Adjust the sensor's alert sensitivity (See guide: [How to Adjust Sensor Alert Sensitivity](#))
- Pause the sensor's notifications (See guide: [How to Pause Sensor Notifications](#))

Related Guides

[How to Add Additional Sensors](#)

[How to Pair AbiBird Sensor with App and Account](#)

[How to Name Sensor Location and Place in Position](#)

[How to Set Up or Modify Daily Routines](#)

[How to Check Sensor Status and Rename Sensor](#)

[How to Adjust Sensor Alert Sensitivity](#)

[How to Pause Sensor Notifications](#)