



This guide is for Android 5+ Smartphone users.



AbiBird®

How to


Add Additional Sensors

You can add more than one sensor to your AbiBird account – perhaps you want to monitor more than one room in your cared-for person’s home.

To add additional sensors to your AbiBird account, you need to:


1. Access My Sensors page
2. Add a sensor
3. Pair sensor with app and account
4. Name sensor location and place in position
5. Set up daily routines

Step 1: Access My Sensors page

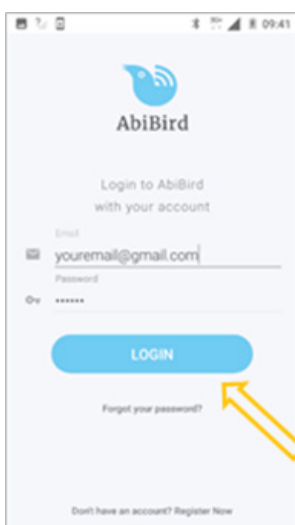

Login to AbiBird with your username (email) and password.
The *My Sensors* page will open.

NOTE: If you are already in the AbiBird app you can access the *My Sensors* page by selecting the **Back Arrow** (top left) repeatedly until the *My Sensors* page opens.

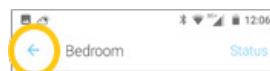




This guide is for Android 5+ Smartphone users.



AbiBird

Step 2: Add sensor

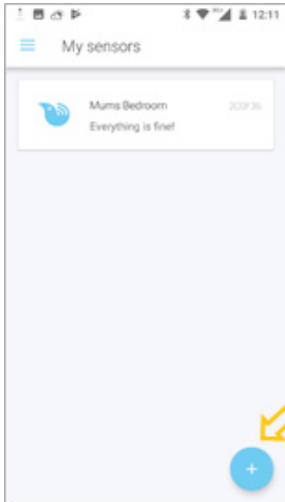


On the *My Sensors* page, select the + icon (bottom right).
The *Waiting for device* page will open.

Step 3: Pair sensor with app and account



To pair the sensor with your app and account, follow the steps in the guide: [How to Pair AbiBird Sensor with App and Account](#).

Step 4: Name sensor location and place in position

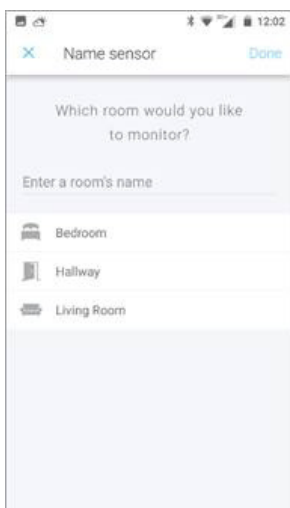

Once you have paired the sensor with the app and your account, you will be prompted to name it and place it in position.

To name the sensor and position it in the room you want to monitor, follow the steps in the guide: [How to Name Sensor and Place in Position](#).




This guide is for Android 5+ Smartphone users.



AbiBird


Step 5: Set up daily routines

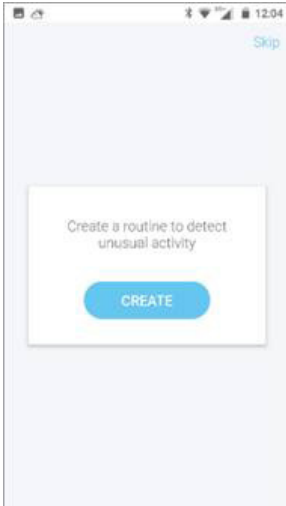

Once you have named the sensor and placed it in position, you will be prompted to set up the expected daily routines for the room.

To set daily routines for the sensor, follow the steps in the guide: [How to Set Up or Modify Daily Routines](#).

That's it

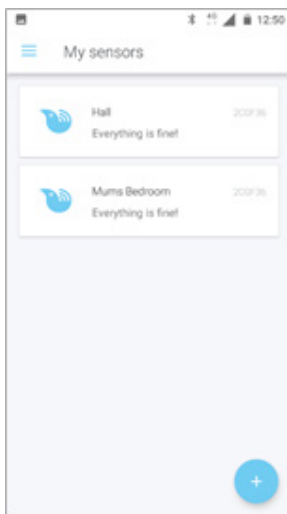

Another sensor has been added to your AbiBird account.

Working in the app, you have:

- Accessed the *My Sensors* page
- Added another sensor
- Paired the sensor and app
- Set daily routines for the sensor

NOTE

You don't set notification preferences for each sensor. Notification preferences apply to all sensors paired to your AbiBird account. However, you can pause notifications for a particular sensor (see guide: [How to Pause Sensor Notifications](#)).

Next Step

Once you have added a sensor you can:

- Check its status and rename it if required. (See guide: [How to Check Sensor Status and Rename Sensor](#))
- Modify its daily routine (See guide: [How to Set Up or Modify Daily Routines](#))
- Adjust its alert sensitivity (See guide: [How to Adjust Sensor Alert Sensitivity](#))
- Pause its notifications (See guide: [How to Pause Sensor Notifications](#))



This guide is for Android 5+ Smartphone users.



AbiBird®

Related Guides

[*How to Pair AbiBird Sensor with App and Account*](#)

[*How to Name Sensor Location and Place in Position*](#)

[*How to Set Up or Modify Daily Routines*](#)

[*How to Check Sensor Status and Rename Sensor*](#)

[*How to Adjust Sensor Alert Sensitivity*](#)

[*How to Pause Sensor Notifications*](#)