



This guide is for iOS 10+ iPhone users.



AbiBird®

# How to

## Add Additional AbiSensors

You can add more than one AbiSensor to your AbiBird account – perhaps you want to monitor more than one room in your cared-for person's home.

To add additional AbiSensors to your AbiBird account, you need to:

1. Access *Devices* page
2. Add AbiSensor
3. Connect AbiSensor with app and account
4. Name and place AbiSensor in position
5. Set up daily routines for AbiSensor

### Step 1: Access *Devices* page

AbiBird

Create an AbiBird account to get started with AbiBird

Name

Email

Password

By tapping create you agree to AbiBird


Terms of Use

CREATE

Already have an account? Log in

Login to your AbiBird app with your username (email) and password. The *Devices* page will open.

**NOTE:** If you are already in the AbiBird app you can access the *Devices* page by paging back (arrow < top left) until the *Devices* page opens.

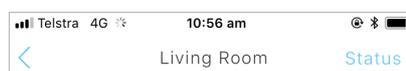




This guide is for iOS 10+ iPhone users.




## Step 2: Add AbiSensor

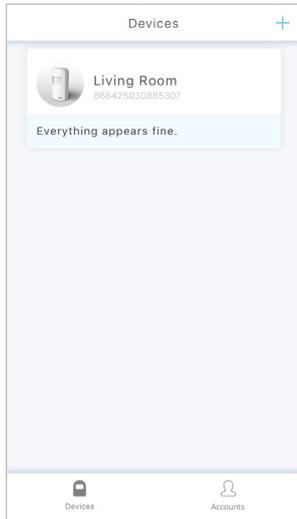


On the *Devices* page, select the + icon (top right).  
The *Searching for AbiBird sensor* page will open.

## Step 3: Connect AbiSensor with app and account

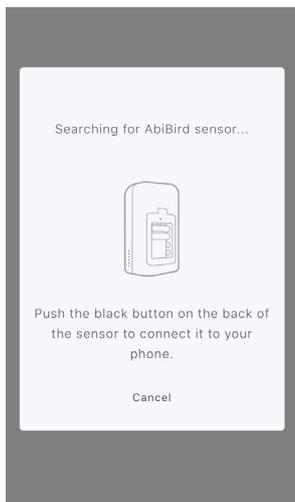

To connect the AbiSensor with your app and account, follow the steps in the guide: [How to Connect AbiSensor with App and Account.](#)

## Step 4: Name and place AbiSensor in position

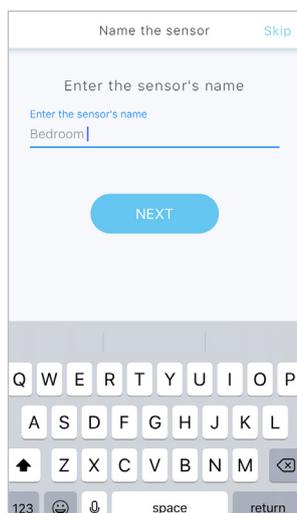

Once you have connected the AbiSensor with the app and your account, you can name it and place it in position.


To name the AbiSensor and place it in the room you want to monitor, follow the steps in the guide: [How to Name and Place AbiSensor.](#)



This guide is for iOS 10+ iPhone users.



## Step 5: Set up daily routines for AbiSensor

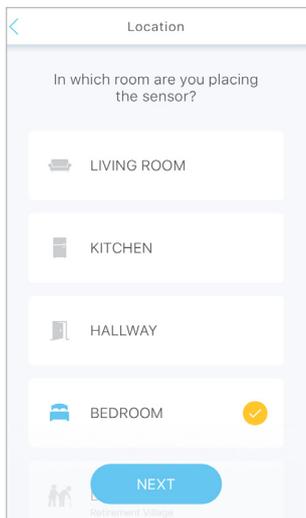

Once you have named the AbiSensor and placed it in position, you can set up the expected daily routines for the room.

To set daily routines for the AbiSensor, follow the steps in the guide: [How to Set Up or Modify AbiSensor Daily Routines.](#)

## That's it!



Another AbiSensor has been added to your AbiBird account.

**NOTE:** The AbiSensor you have just added will show as 'inactive' until the AbiBird app updates the AbiSensor's status, which will take about an hour. However, you can test if the AbiSensor is working by triggering a 'Request for contact', see the guide: [How to Use AbiSensor to Trigger a Request for Contact.](#)

Working in the app, you have:

- Accessed the *Devices* page
- Added another AbiSensor
- Connected the AbiSensor and app
- Named and placed AbiSensor in position
- Set daily routines for the AbiSensor

### NOTE

You don't set notification preferences for each AbiSensor. Notification preferences apply to all AbiBird devices connected to your AbiBird account. However, you can pause notifications for a particular AbiSensor (see guide: [How to Pause AbiSensor Notifications](#)).



This guide is for iOS 10+ iPhone users.



AbiBird®

## Next Step

Once you have added an AbiSensor you can:

- Check its status and rename it if required. (See guide: [How to Check AbiSensor Status and Rename AbiSensor](#))
- Modify its daily routine (See guide: [How to Set Up or Modify AbiSensor Daily Routines](#))
- Adjust its alert sensitivity (See guide: [How to Adjust AbiSensor Alert Sensitivity](#))
- Pause its notifications (See guide: [How to Pause AbiSensor Notifications](#))

## Related Guides

[How to Connect AbiSensor with App and Account](#)

[How to Name and Place AbiSensor](#)

[How to Set Up or Modify AbiSensor Daily Routines](#)

[How to Check AbiSensor Status and Rename AbiSensor](#)

[How to Adjust AbiSensor Alert Sensitivity](#)

[How to Pause AbiSensor Notifications](#)

[How to Use AbiSensor to Trigger a Request for Contact](#)