



This guide is for all Smartphone users.



AbiBird®

How to


Use AbiBird Sensor to Trigger a Call for Help

Your cared-for person can use the AbiBird sensor to trigger a 'Call for Help' at any time.

To trigger a 'Call for Help' alert



Continuously tap or shake the sensor firmly (to activate the vibration sensor) and wait until it beeps and the light turns red.

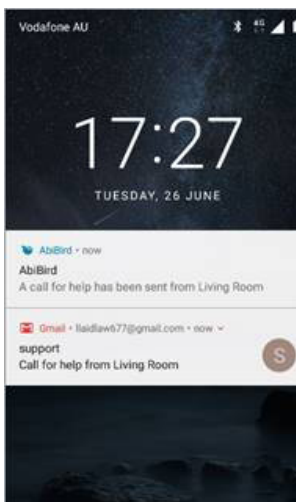

An alert notification and email will be sent to the Carer's phone...



This guide is for all Smartphone users.



AbiBird®



... and the sensor's graph will show a 'Call for Help' alert (one or more yellow dots).