



This guide is for Android 5+ users.



AbiBird®

How to


Change Timezone Settings

By default, the timezone settings on the app are synced with your current location. You are able to change the settings to reflect the location of the AbiBird sensor, if you are not in the same timezone.

To set modify a sensor's timezone settings, you need to:

1. Access Routine page
2. Access a scheduled routine
 - a. Select Advanced Settings
 - b. Change timezone

Step 1: Access 'Routine' page

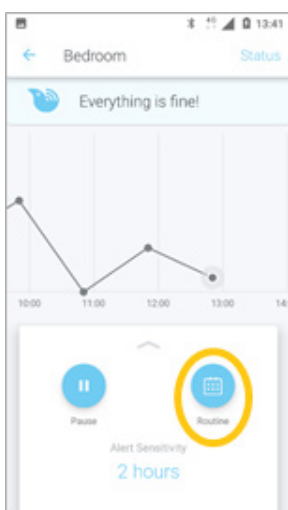

From the *My sensor* page, click on the sensor for which you would like to view the routine. The sensor's *graph* page will open.

From the panel at the bottom of the *graph* page, select the **Routine** icon.




This guide is for Android 5+ users.



AbiBird®


Step 2: Access a scheduled routine

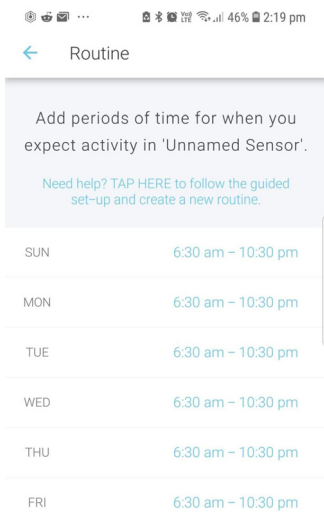

On the **Routine** page, select any scheduled time period.

The *Edit Time Slot* page will open.

Step 2a: Advanced settings

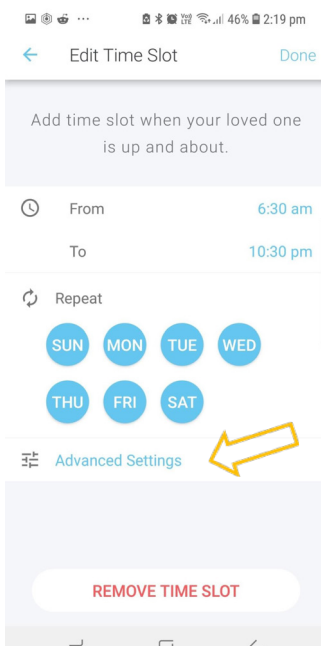


On the **Edit Time Slot** page, select **Advanced Settings**.

The current *Timezone* will appear below.




This guide is for Android 5+ users.

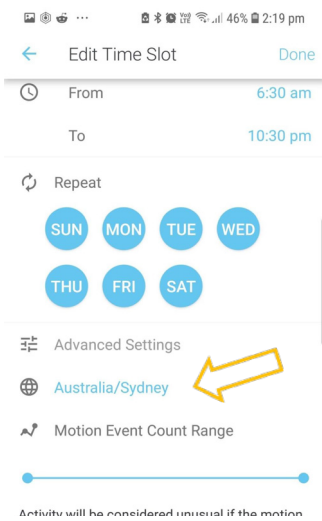

Click on the current *Timezone* to edit.
The dropdown menu will open.

The example shows the Timezone of the person who has the app is set for Sydney.

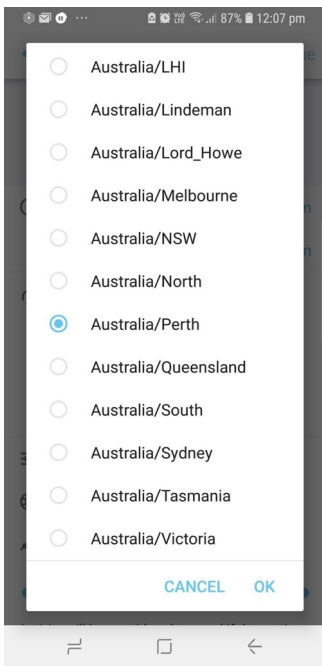


Select the new Timezone for the location of the AbiBird sensor from the dropdown menu. Press *OK*.
The **Edit Time Slot** screen will open. Press *Done*.

The example shows the Timezone of the cared-for person, who has the AbiBird sensor, is now set to their home location of Perth.

This enables the to Carer to view the chart in actual time from anywhere in the world.

That's it

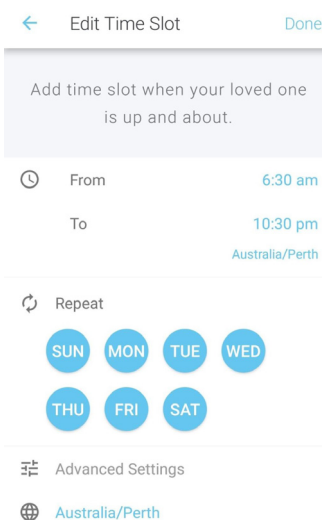

Timezones have been changed for the sensor.

NOTE

If you have more than one sensor, you will need to change the Timezones as detailed above for each sensor.