



This guide is for iOS 10+ iPhone users.



AbiBird®

# How to

## Change Timezone Settings


By default, the timezone settings on the app are synced with your current location. You are able to change the settings to reflect the location of the AbiBird sensor, if you are not in the same timezone.

To set modify a sensor's timezone settings, you need to:

1. Access Routine page
2. Access a scheduled routine
  - a. Select Advanced Settings
  - b. Change timezone

### Step 1: Access 'Routine' page

The way you access the *Routine* page will depend on whether you are setting up AbiBird for the first time or modifying existing settings.



If...	Then...	
You are setting up AbiBird for the first time...	The <i>Routine</i> START page opens after you have named your sensor location. (See guide: <i>How to Name Sensor Location and Place in Position</i> ) <ul style="list-style-type: none"> <li>• Select the <b>START</b> button.</li> </ul> The <i>Routine</i> time-setting page will open. [Go to Step 2]	




This guide is for iOS 10+ iPhone users.



AbiBird®

If...	Then...	
You have already set up AbiBird and want to modify the existing settings...	<ul style="list-style-type: none"><li>• Login to AbiBird with your username (email) and password. The sensor menu opens.</li><li>• Select the sensor for which you want to modify settings. The sensor's graph page will open.</li></ul>	
	<ul style="list-style-type: none"><li>• From the panel at the bottom of the sensor's graph page, select <b>Routine</b> icon. The <i>Routine</i> page will open. [Go to Step 2]</li></ul>	

## Step 2: Access a scheduled routine

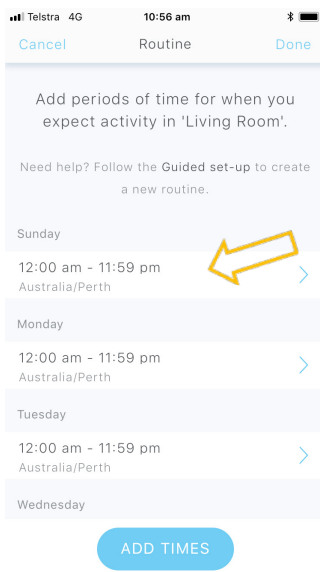

On the **Routine** page, select any scheduled time period.  
The *Time Period* page will open.




This guide is for iOS 10+ iPhone users.




AbiBird

## Step 2a: Advanced settings

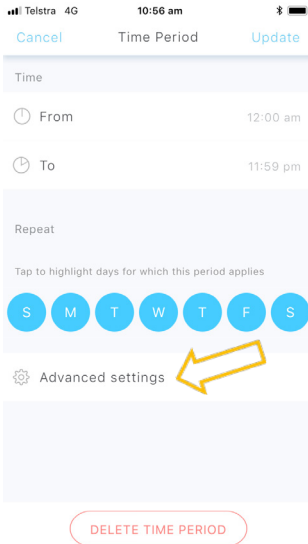

On the **Time Period** page, select **Advanced settings**.

The *Advanced settings* page will open.

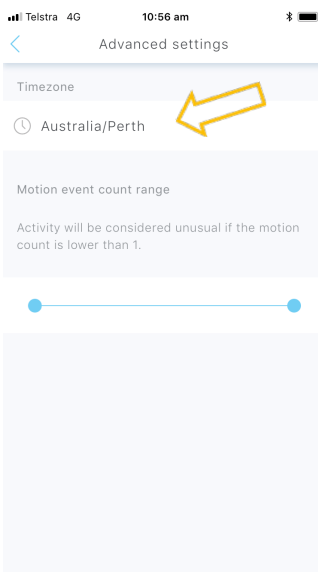


The **Advanced settings** page shows the current Timezone.

Select the Timezone.

The *Timezone* page will open.

*The example shows the Timezone of the person who has the app is set as Perth.*

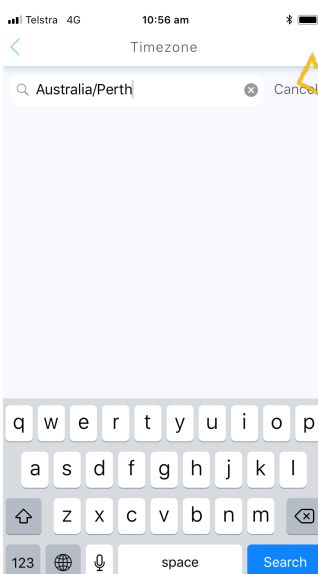


On the *Timezone* page, press **Cancel** to remove current setting.

A dropdown menu opens showing the timezone options.




This guide is for iOS 10+ iPhone users.

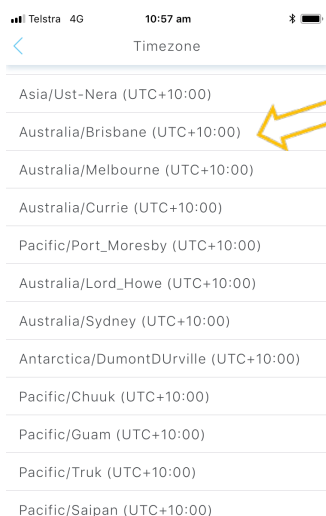

Select the new Timezone for the location of the AbiBird sensor from the dropdown menu.

The *Advanced settings* page will open.

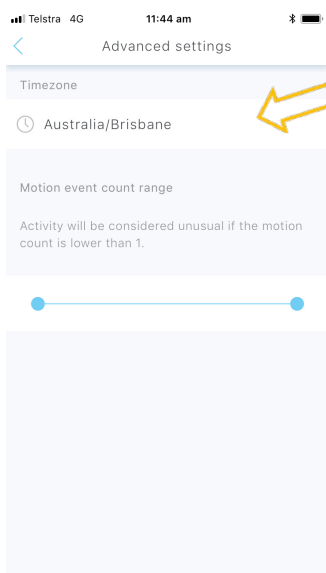

The **Advanced settings** page shows the updated timezone.

Press the arrow at top right of screen to return to **Time Period** page, then select *Update* (top right)

The **Routine** page will open

Select *Done* (top right)

*The example shows the Timezone of the cared-for person, who has the AbiBird sensor, is now set to their home location of Brisbane.*

*This enables the to Carer to view the chart in actual time from anywhere in the world.*

## That's it



Timezones have been changed for the sensor.

### NOTE

If you have more than one sensor, you will need to change the Timezones as detailed above for each sensor.